

OPERATING INSTRUCTIONS

**OLYMPIC
ULTRA-LITE™**
Catalog #50601



Intended Use

Transporting small animals, weighing up to 250 pounds, in veterinary clinics and for brief outdoor use for emergency transport.

Placing Stretcher on Cart

Place the 4 corners of the stretcher over the 4 vertical posts **A** and lower the stretcher until it rests securely on the cart. Each post should extend about 2 inches above the stretcher.

Foot Brakes

To lock foot brake, step on end of lever **B**. To release brake, step on other end of lever.

CAUTION: Always lock foot brakes when removing or replacing stretcher or patient.

CAUTION: Always make sure that all 4 foot brakes are unlocked before moving the cart.

Taking X-rays Through the Stretcher

Place stretcher with patient on the X-ray table. CAUTION: Make sure all 4 feet on bottom of stretcher rest securely on the table.

Other Safety Cautions

- When pushing the cart always grip the two handles **C** at either end to avoid tipping. Do not roll the cart by pushing on the stretcher.
- Never roll or turn the cart so fast as to create a danger of tipping. Push slowly and carefully.
- Be careful not to tip cart when rolling over obstructions such as thresholds, electrical cords, uneven floors, or outdoors.
- When necessary, strap patients to the stretcher using the two Velcro straps **D**. Straps can be used parallel or in X-cross pattern.
- Never leave a patient unattended on the cart.

Cleaning

Wipe stretcher and cart with ordinary hospital disinfectants. Avoid getting liquid on casters.

Maximum Load Capacities

Cart and stretcher – 250 pounds

For further information contact

OV OLYMPIC VETERINARY

1-800-789-6867 www.OlympicVeterinary.com